

Mini Close / Mini Pad

Appliances for Bathing


Instruction Manual

You can contact us at
E-mail : inquiry@alcare.co.jp Fax : +81-3-5611-7827
www.alcare.co.jp/e

- Prior to use, carefully read the instruction manual so that you understand the product's features.
- Keep the instruction manual close at hand and read it again when needed.


Types and Features

Mini Close

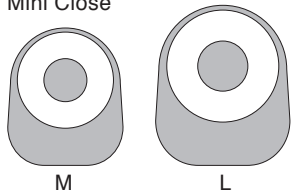

- This is a closed type ostomy bag whose bottom side is wide. You can use it even when there is small amount of faeces or urine.
- For your convenience, there is 40 mm precut hole on it.

Mini Pad

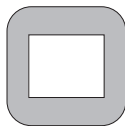

- The surface remains soft, but it is completely-waterproof type, so you can take a bath without ingress of the water.
- Highly water absorbent pad can absorb intestinal fluids and protect stoma.

Application

- ① Clean the skin around stoma with steaming towel. Dry well.
 - * Make sure to remove any oil thoroughly from the surface of the skin.
- ② Peel off the release paper from adhesive face. Adjust the hole of Mini Close or the pad part of Mini Pad to the stoma, and apply it from bottom to top.
 - * Any wrinkles on adhesive side may cause leakage of faeces, so be careful with it.
- ③ Press the adhesion area against the skin around the stoma to ensure its adhesion.
 - * Warm the adhesive face before application if it is cold.
- ④ Exchange when the faeces or urine are accumulated.
 - * Slowly remove the ostomy bag while you are placing your other hand on your skin. Do not remove suddenly since it may cause skin trouble.

Caution

- Prior to use, carefully read the instruction manual.
- Do not store in an area of high-temperature or high-humidity area or in a freezer.
- Do not use for irritated skin.
- Use carefully if the skin is sensitive.
- Stop using if you have any skin trouble.
- These products are ostomy appliances just for ostomate use. For your safety, please do not use for other purposes.

ALCARE Co., Ltd.

19F, Arcacentral Bldg., 1-2-1 Kinshi,
Sumida-ku, Tokyo, 130-0013, Japan
<http://www.alcare.co.jp/e>