

Cellcare 2-BC Youcare 2-BC Procure 2-BC

2-Piece Ostomy System / Bath Cap

Instruction Manual

You can contact us at
E-mail : inquiry@alcare.co.jp Fax : +81-3-5611-7827
www.alcare.co.jp/e

- Prior to use, carefully read the instruction manual so that you understand the product's features.
- Keep the instruction manual close at hand and read it again when needed.

Application

● How to Apply

- ① Remove the ostomy bag before taking a bath or a shower. (For the way to remove the ostomy bag, please see the Instruction Manuals putting in our product packages of faceplate or ostomy bag.)
- ② Apply a Bath Cap to the faceplate adhered to the body. Please apply Bath Cap to faceplate in the same way as ostomy bag.
- ③ After applying the Bath Cap, trace the ring part to make sure the Bath Cap is securely connected with the faceplate.



● How to Remove

- ① After taking a bath or a shower, hold the tab for removal on the faceplate by one hand and pinch the tab for removal on the Bath Cap to slowly remove the cath cap. (Some faeces may be adhered inside of the Bath Cap. Be careful not to soil anything with the faeces when you remove.)
- ② After removal of the Bath Cap, apply the ostomy bag as usual.
- ③ Clean the used Bath Cap with lukewarm water, and dry well. (The exceeding heat by a hair-dryer may make the Bath Cap deformed. Do not use a hair-dryer and let it dry naturally to keep quality.)

Precautions for Stock

Avoid the followings to keep quality:

- Storage under the condition of high temperature (exceeding 40°C) and high humidity.
- Storage in direct sunlight.
- Storage with excessive weight.

ALCARE Co., Ltd.

19F, Arcacentral Bldg., 1-2-1 Kinshi,
Sumida-ku, Tokyo, 130-0013, Japan
<http://www.alcare.co.jp/e>