

Deofine Powder

Powdery Deodorant for Ostomates

Instruction Manual

- Prior to use, carefully read the instruction manual so that you understand the product's features.
- Keep the instruction manual close at hand and read it again when needed.

You can contact us at
 E-mail : inquiry@alcare.co.jp Fax : +81-3-5611-7827
www.alcare.co.jp/e

《Deofine Powder》 is a powdery deodorant for ostomates. For safety, use it in accordance with this Instruction Manual and not for any other purposes. If you have any inquiries, contact us by e-mail, fax, or our distributor in your area.

Precautions



Warning: Usage of Deofine Powder may result in physical injury.

- This powder should not be swallowed. Take care to ensure that elderly people, the mentally disabled, infants, and children do not accidentally swallow it, as the powder could cause choking.
- This product contains mint perfume. You may feel sick depending on your constitution or physical condition. In such cases, stop use of the product immediately. As pregnant women and children are especially susceptible to fragrances, please consult your physician before use.
- Do not apply to injured skin (skin with wound, swelling, or rash). In case of contact, flush with water immediately.
- If any dermal / mucosal abnormality is observed, such as flare, itching and irritation, immediately stop treatment and contact a physician.
- Avoid contact with the eyes. In case of contact, flush with water immediately.
- Do not inhale through the nose or mouth. In case of inhalation, flush with water immediately.

Caution: Usage precautions


- Avoid touching the adhesive part of the face plate with powdery hands and do not put powder on it. Otherwise, the face plate may lose adhesion and fall off.
- Avoid touching the coupling part or the bag closure (e.g., clip or cap) of the ostomy bag. Otherwise, you may feel discomfort when handling or the coupling part or bag closure may become loose.
- Do not use the powder with any other products that are intended to be used in the ostomy bag. Otherwise, the deodorizing effect may weaken.
- Use the powder once then throw the remained powder away as burnable garbage in accordance with local disposal rules. If left unsealed, the powder will absorb moisture, which may significantly reduce the deodorizing effect and adversely affect its quality.
- If the powder attaches to clothing, immediately brush off and wash with a general detergent.

Instruction for Deofine Powder

- Open the package with hands or scissors.
- Avoid touching the adhesive part of the face plate or the coupling part of the ostomy bag. For a 2-piece ostomy system, the powder should be put in through the coupling part or the open end of the ostomy bag. For a 1-piece ostomy system, the powder should be put from the stoma hole of the face plate or the open end of the ostomy bag.
- Open a new package and put the powder in the ostomy bag each time after discharge of excrement.


Before Application

● For 1-piece

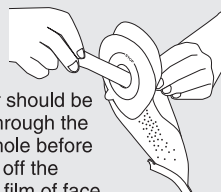

Powder should be put in through the stoma hole before peeling off the release film of face plate.

● For 2-piece

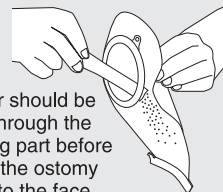


Powder should be put in through the coupling part before setting the ostomy bag onto the face plate.

Caution: For an open-ended ostomy bag, make sure that the open end is closed.

During Application

When adding new powder after discharge of excrement, hold the lower open end of the ostomy bag at an upward angle and insert the front edge of the package into the middle of the bag.

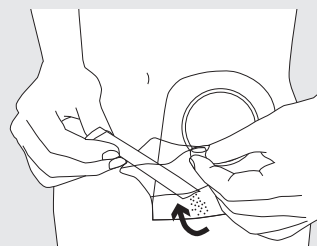

Caution: Avoid allowing the powder to touch or remain around the open end of the ostomy bag. Otherwise, the bag closure (e.g., clip or cap) may become loose, causing the bag to leak or drop off.

Storage and Handling

- Do not transfer the powder to any other containers. Otherwise, it may be used improperly or absorb moisture, which may adversely affect its quality.
- Keep away from infants and children.
- Do not put anything heavy on the package or box. Otherwise, the package may be damaged or broken.
- Avoid direct sunlight, excessive heat and moisture.

Disposing of Deofine Powder

- The package is made of plastic. Dispose of it according to local rules for separated garbage disposal.

Specification

Type	Product Code No.	Net Weight	Contents Per Box
—	17891	1.5g / Pack	30 packs

ALCARE Co., Ltd.

19F, Arcacentral Bldg., 1-2-1 Kinshi, Sumida-ku, Tokyo, 130-0013, Japan
<http://www.alcare.co.jp/e>

International Sales Div. TEL: +81-3-5611-7819 / FAX: +81-3-5611-7827