

# Procare Powder

Powder Type of Skin Barrier


## Instruction Manual

You can contact us at  
E-mail : inquiry@alcare.co.jp Fax : +81-3-5611-7827  
www.alcare.co.jp/e


- Before use, carefully read the instruction manual so you understand the product's features.
- Keep this instruction manual close at hand and refer to it again when needed.

**Introduction** This product is a powder-type skin barrier that fills the space between the faceplate and the stoma.

## How to use (1):

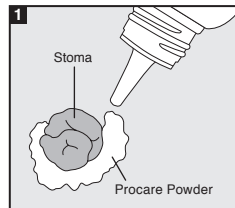


1 Sprinkle the powder on the area where you want to apply.

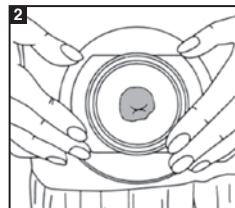

2 Remove excess powder. Apply the ostomy appliance as usual.

## How to use (2):

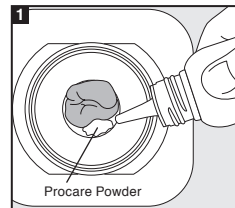

1 If there is any space between the faceplate and the stoma after application of the ostomy appliance, fill it with powder.

## Precautions for Use

**Warnings (precautions for improper use that may lead to serious bodily injury)**

- This product is not food. To avoid accidental swallowing, this product should be stored appropriately to keep it out of reach of elderly people, people with dementia, infants and children.

### Precautions

- If you notice any problems on your skin, stop using the

- product immediately and consult a doctor.
- After use, seal the product tightly and store it at room temperature.
- Wipe off any Procure Powder that remains on your skin.

## Precautions for storage

- Do not store the product at high temperature and humidity, or at low temperature.

**ALCARE Co., Ltd.**

19F, Arcacentral Bldg., 1-2-1 Kinshi,  
Sumida-ku, Tokyo, 130-0013, Japan