

# Procure Remover

## Ostomy Appliance Remover

### Instruction Manual


You can contact us at  
E-mail : [inquiry@alcare.co.jp](mailto:inquiry@alcare.co.jp) Fax : +81-3-5611-7827  
[www.alcare.co.jp/e](http://www.alcare.co.jp/e)

- Prior to use, carefully read the instruction manual so that you understand the product's features.
- Keep the instruction manual close at hand and read it again when needed.


### Application



① Tear the aluminum pack at the pre-cut part, and take a remover out by hand.

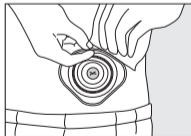

② First pick up the upper end of stoma appliance adhered to the skin. Wipe the gap between the skin and the appliance with a remover, and remove the appliance slowly.

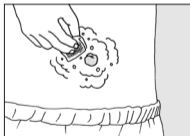

③ After removing the stoma appliance, wash well the skin with soapy water and rinse. After rinsing, wipe well by a dry gauze or a towel.

In the sections of [Precautions for Use] and [Precautions for Stock], the degree of risk is defined as follows:

- ⚠ Warning ..... May cause physical injury
- ⚠ Caution ..... May cause property damage

### Precautions for Use

#### ⚠ Warning

- Do not use for the part of wound, swelling, eczema etc.
- Keep away from fire, or it may catch fire.
- After tearing the aluminum pack, use up within 24 hours, or it may cause any skin trouble due to an oxidized ingredient liquid.
- After use, wash well the skin with soapy water and rinse. After rinsing, wipe well by a dry gauze or a towel. Inadequate wiping off the liquid on the peristomal skin, a remained oil may cause skin troubles or may cause adhesion of the appliances weak.
- After use, throw it away immediately. Children may put into mouths because of its orange flavor. If children would eat it, immediately see a doctor.
- The liquid gets in the eyes, wash the eyes with running water, and see a specialist.
- When you find any skin trouble, stop using immediately, and see a specialist.

#### ⚠ Caution

- If the liquid is soaked through the clothes, immediately dip into water and wash as usual.

### Precautions for Stock

#### ⚠ Warning

- Keep away from fire. As Procure Remover involves oil, it may catch fire if it places near fire.
- Keep away from children.

#### ⚠ Caution

- Store in a room-temperature. Aluminum pack may be damaged when exceeding 40°C.

**ALCARE Co., Ltd.**

19F, Arcacentral Bldg., 1-2-1 Kinshi,  
Sumida-ku, Tokyo, 130-0013, Japan  
<http://www.alcare.co.jp/e>