

# HN-2 / Extremities Report


**Centre Saint-Jean  
Saint-Doulchard**

## <Challenge>

For upper and lower extremities treatments, there is some uncertainty about repositioning, as we have no restraint adapted to this type of mobile location.

## <Application>

For extremities, upper and lower we use Moldcare HN-2 (20 x 45 cm), sometimes BR-3 (60 x 60 cm). The Moldcare is molded in this case to the arm and the hand to allow for customized repositioning. The axes of the lasers are marked on it, enabling the treatment area to be aligned very precisely. It also limits differences in table value from one day to the next if the Moldcare is molded to an indexed contention.

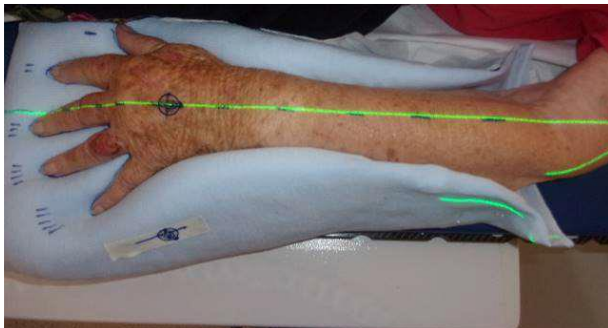

## <Result>

Excellent results.

Moldcare helps us to reposition all our patients on a daily basis.

It enables us to optimize registration and reproducibility, particularly for complex treatments like extremities.